


**NORTHWEST  
COMMUNITY  
EMERGENCY  
MEDICAL  
SERVICES  
SYSTEM**



EST 1972

DIANA:new-CPAP-dec-2011


**NEW**

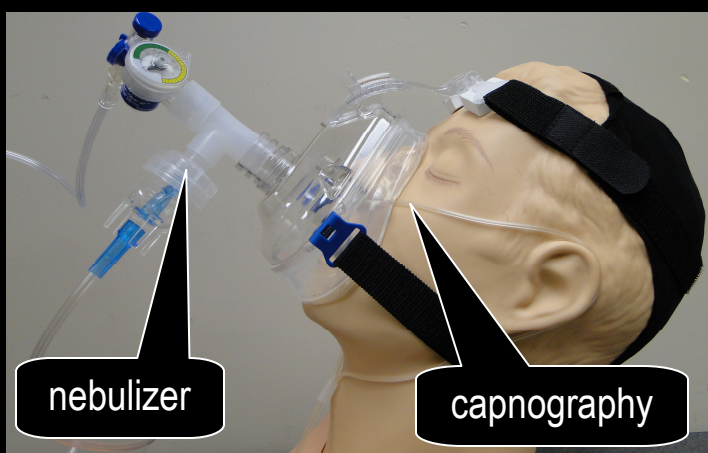

nebulizer

capnography

# Flow-Safe CPAP

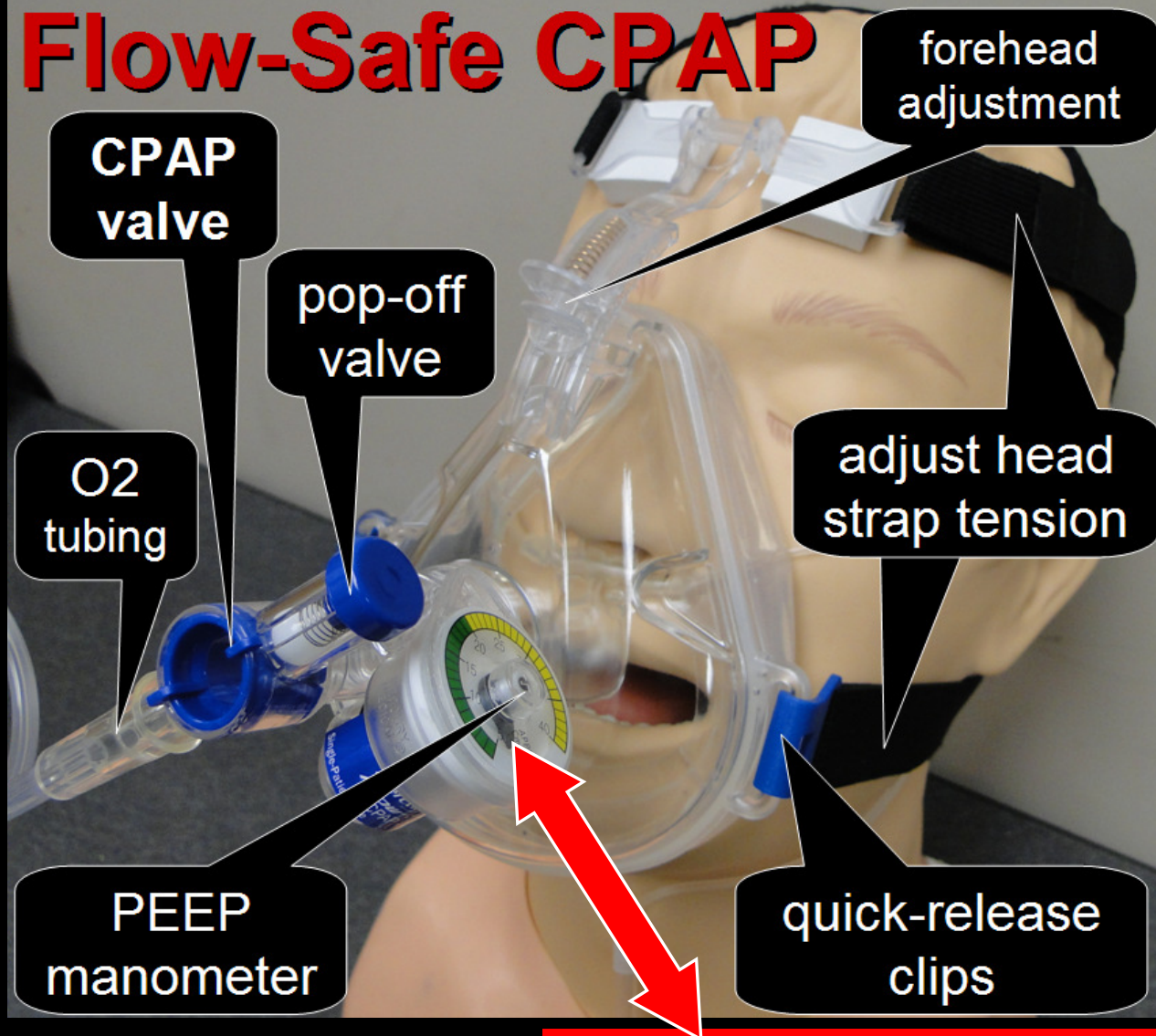

CPAP  
valve

pop-off  
valve

O2  
tubing

PEEP  
manometer

forehead  
adjustment

adjust head  
strap tension

quick-release  
clips

**O<sub>2</sub> L flow = PEEP (cm H<sub>2</sub>O)**

10 L = ~ 1.5 - 2 cm H<sub>2</sub>O PEEP

15 L = ~ 3 - 4 cm H<sub>2</sub>O PEEP

20 L = ~ 6 - 7 cm H<sub>2</sub>O PEEP

25 L = ~ 8.5 - 10 cm H<sub>2</sub>O PEEP

**Manometer between CPAP valve & mask  
measures actual PEEP delivered.  
Leak (poor face-mask seal) will affect PEEP.**

