


# Flow-Safe® CPAP



Fold & Store w/ CPAP

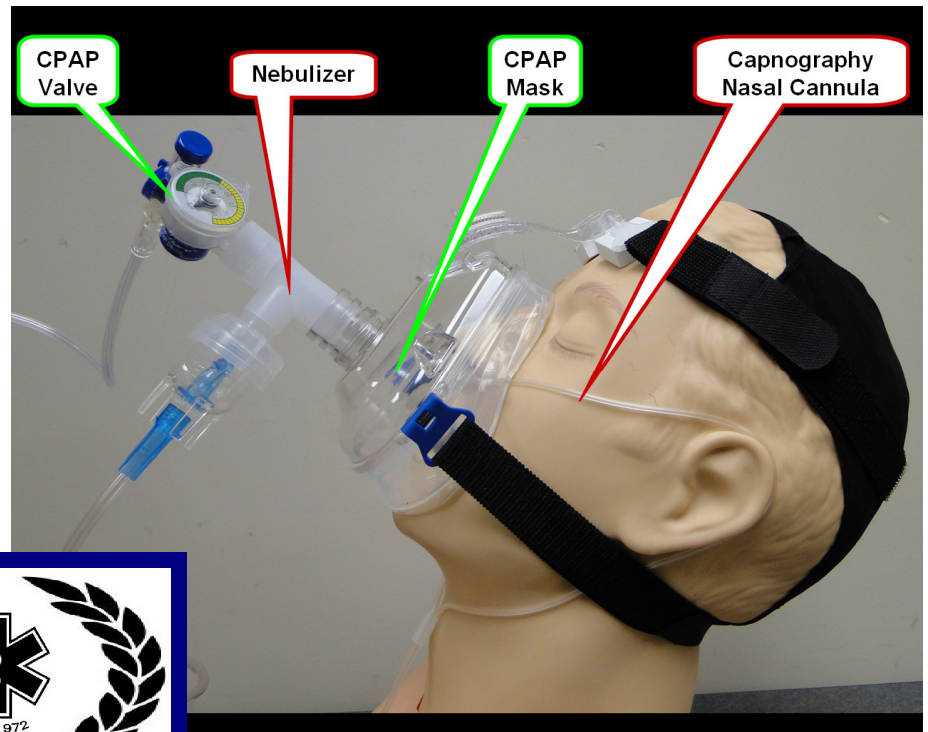

## Connect Nebulizer Between CPAP Valve & Mask



DIANA:cpap-neb-mar-13

## Some Nebulizers Require Adapter (same adapter used for adv airway in-line neb)



Universal Cuff Adapter  
22mm ID connections  
catalog no. HUD 1421

Universal Cuff Adaptor  
• Standard 22mm I.D connections  
• 50 per pack  
Product: Universal Cuff Adaptor  
Catalogue No: HUD1421